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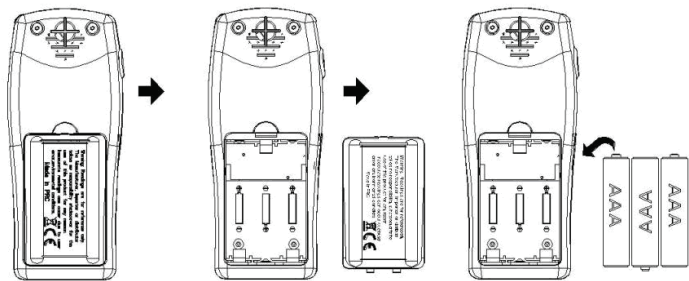
Measurement



- Ritmix RAT-701 Personal Breathalyzer, is a hand held Professional Alcohol Analyzer. Using the electrochemical reaction for platinum vs ethanol molecules, Ritmix RAT-701 Personal Breathalyzer can give stable and reliable measuring technology.
- Wash your mouth with water before using. Abstain from smoking or drinking alcohol for at least 20 minutes.

Power Supply

- Slide back the battery cover on the back of the device.
- Install 3 x 1.5V alkaline batteries (AAA) into the battery compartment.
- When battery energy not enough to do the alcohol testing, battery inductor flashes. Please replace with a new battery immediately.




Parts

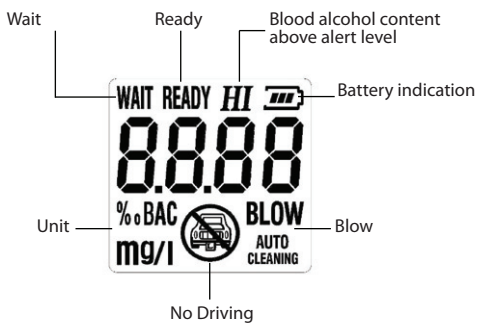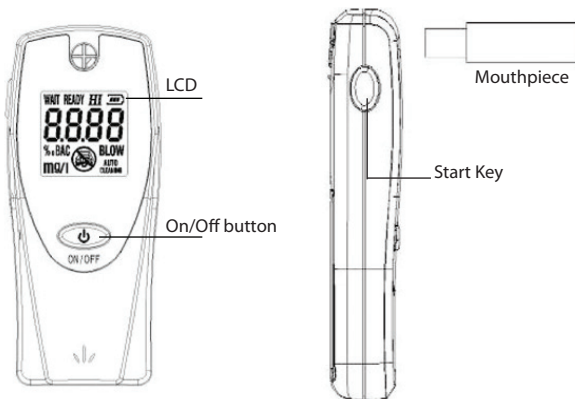Operation

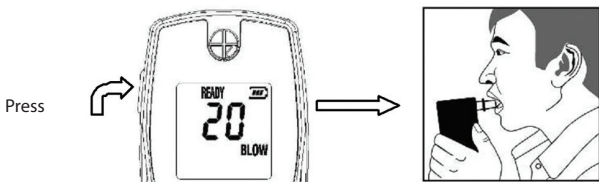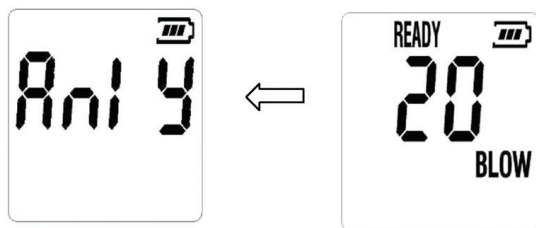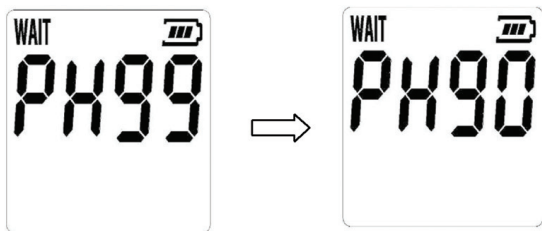- Insert a mouth piece and then press the **ON/OFF** Key for 1 second to turn on/off the power. When the machine is on, "**ALCO**" will be shown on the display. Press **ST** key and sensor pre-heating will be activated. "**PH**" flashes for 3 times and start to count down ("**PH99**" counts down "**PH00**"). "**READY**" & "**BLOW**" flashes alternately and count down for 30 seconds together with beep tones when the machine is ready to take sample.
- When ready, user should blow continuously through the mouth piece for at least 7 seconds while press the **ST** key to activate the sampling system. Auto detect your breath sample and stop blowing after you hear a "beep" tone. If the test sample is successful, "**Anly**" (indicates analyzing) will flash and the reading will be displayed accordingly.
- User can press **ST** key for another test. When the display counts down to zero and the "**READY**" indicator shows on the display, a beep sounds to begin testing again.
- Your test result will be displayed on the LCD within **10 seconds**.

Important Notes

- When unused for a long time or at a low temperature, the machine will take longer to warm up time.
- After 5 consecutive tests, allow the device to rest for 2 minutes.
- Please store in a dry environment at room temperature.
- Avoid extreme heat or long term sun exposure.
- When battery indicator flashes, replace with a new battery.
- If the difference between two successive readings is $>0.1\%$ (0.01%) user **should** test again.
- During the breath sampling, do **NOT** block the outlet.
- An "Err" flashes on the display for 5 seconds when user fails to start the test within 5 seconds.

Specification

Fuel cell Sensor.

Display Range: 0.00~ 4.00 ‰ BAC; or 0.00~0.40% BAC or 0.00mg/L~2.0mg/L (convection factor is 2.0)

Power Input: 3 x AAA alkaline battery.

Display: Digital LCD display with bi-colour Backlighting.

Auto Power Off: 2 minutes.

Breath Sampling: about 7 seconds continuous breath sample at 20 L/min flow rate.

Dimension: 107 x 45 x 18 mm

Weight: 55 g

Operating temperature: 10 ° to +40 °, for optimum operation

Agency Approval: CE

Accessory: 2 pcs mouthpieces

Alert Signals

DISPLAY & ALERT	RESULT (BAC)
Green backlight and shows 0.00 ‰	$\leq 0.05\%$ BAC (0.005% BAC or 0.025 mg/l)
Alarm: "NO DRIVING" sign flashes with alarm and red backlight for 16 seconds, Show the digital reading on the display	0.05 ‰~3.99 ‰ BAC (0.005%~0.399% BAC or 0.025~1.995mg/L)

Precautions

- Your blood alcohol content (BAC) may change within minutes
- Do not use this device to determine if you can drive.
- Do not share with friends. This unit is designed for personal use for accuracy and hygienic considerations.
- After drinking, it is recommended to last after 20 minutes, it is because 20 minutes needed approximately for alcohol to be absorbed into blood from the digestive organs, and residual alcohol remaining in the mouth takes this long to dissipate.

For accurate result, do not repeat next testing within 3 minutes after last time.

- Do not smoke and eat for 20 minutes prior testing.
- Do not use chemicals to clean the tester directly. For cleaning the mouth piece please use the dry material or paper napkin.
- Do not blowing liquid into the breath pipe.
- Do not use any excitant substances near the tester, such as paint, insecticides, alcohol.
- Do not place the tester in the closed environment with contaminated air.
- Each individual's body responds differently to alcohol. Drink one glass of wine somebody can feel slightly drunken, somebody can feel heavily drunken.
- Diabetes or low-calorie diet or other factors are leading to the rising of the concentration of ketones in exhalation and can cause false positive results.
- Any of Alcohol testers doesn't give the 100% of guarantee of the accurate results. The most accurate results give blood measuring test. That's why if the road patrol has stopped you, and alcohol tester showed the presence of alcohol in blood, require the blood test.
- Reports of Worlds Health Organization shows that if the alcohol concentration reaches the 0.08 g/ 100 ml, than the risk of road accidents increases in 2.5 times to compare if there is no alcohol in blood. If the alcohol concentration in blood reaches 0.1 g/100 ml, risk of road accidents increases in 4.7 times, than there is no alcohol in blood. Even if the slight quantity of alcohol is drunken it increases risk of road accidents in 2 times.
- The unit is designed to be used as an aid to promote reasonable driving. It should not be used in anticipation of the use or operation of vehicles, machinery, or equipment. The unit makes no references whatsoever to blood alcohol content or other such measures of intoxication promulgated by government, medical or other agencies or bodies. The result of this alcohol gives BAC for reference only; manufacturer does not to take any legal responsibility.
- Test results of alcohol tester are not probative.
- Don't drive in a drunk condition of any degree of intoxication.
- You cannot use the test results in Court.

Warranty

The company-producer guarantees the normal work of device during the all warranty period if the device will be used according to the user manual.

Warranty period for this device is 6 months since the day of the selling through the commercial network. Time of service for this device is not less than 1 year.

The company-producer doesn't accept reclamations in case of the mechanical damage of device, in case of dismantle, wrong usage, or using not in a proper way.

This device will serve you for many years if careful handling.

 Company «Ritmix» has the right to change the Specification without notification.

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